

## CURE MEDICAL PRODUCT CATALOG



“Only Cure Medical offers exceptional products for a better today while helping to ensure an even brighter tomorrow by *donating 10% of net income to research for a cure* for spinal cord injuries and central nervous system disorders. Thank you for *choosing a Cure!*”



*Bob Yant  
Cure Medical founder  
and Cure Catheter® user*

# THANK YOU FOR CHOOSING A CURE!


Request free samples • [www.curemedical.com](http://www.curemedical.com) • 800.570.1778

## Contents

Product	Page
Cure Ultra®	4
Cure Twist®	5
Cure Medical® Pocket Catheter	6
Hydrophilic Cure Catheter®	7
Simple Hydration & Prep Steps	8
Cure Catheter® Closed System Singles	8
Cure Catheter® Closed System Kits	9
Cure Medical® Catheter Insertion Kits	10
Cure Catheter®	11

## Catheter Size Guide


To aid in product selection, the approximate outer dimension of the catheter French (FR) size is shown below. Additionally, the funnel end colors correspond to FR sizes for easy identification.

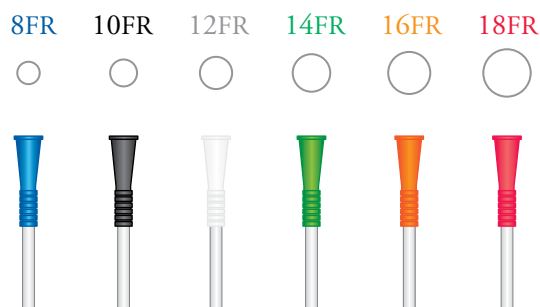

## Cure Medical Product Benefits

### Advancing Medical Research

Cure Medical **donates 10% of net income** to research for a cure for spinal cord injuries (SCI) and central nervous system disorders (CNS/D). Several of the research centers that Cure Medical proudly supports have already achieved a breakthrough in regeneration in the corticospinal tract.

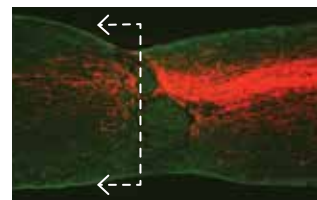

CST nerves in a mouse, (stained red), can be seen growing beyond the lesion site (dotted line). This indicates successful regeneration and offers hope for a cure for humans with CNS/D.

As a result of the Cure Commitment to research, use of any Cure Medical product helps contribute to a promising future.

Supporting research isn't the only Cure Medical distinction. Material choices, unique manufacturing processes, and user driven innovations also set Cure Medical apart from other catheter companies.



Not Made With DEHP/DINP, BPA, or Natural Rubber Latex

Other manufacturers of intermittent catheters often use Di(2-ethylhexyl)phthalate (DEHP) or Diisononyl Phthalate (DINP) as an economical way to make catheters flexible. These phthalates are included in a published list\* of chemicals known to cause cancer and reproductive harm according to the state of California – therefore, Cure Medical products are not made with them.

Natural rubber latex is a chemical which commonly causes irritation and can cause life-threatening allergic reactions. It may be used in other brands of catheters – fortunately, Cure Medical products are not made with it.

By investing in advanced technology and selecting high quality materials for manufacturing, Cure Medical helps users experience safety, comfort, and peace of mind.

\*<https://www.p65warnings.ca.gov>

## Additional Benefits

### Polished Eyelets

Cure Medical uses an ultrasonic process to ensure that all eyelets on all catheters are always exceptionally smooth. The difference in eyelet quality between a Cure Medical product and others can be seen – and felt!

*“When I first tried Cure Medical catheters, I noticed a smoother insertion right off the bat. That's important to me because I just had surgery for the second time to remove strictures.”*

– United Spinal Chapter President and paraplegic Rick Hayden

### User Driven Innovations

Although Cure Medical intermittent catheters function like other brands, they reflect unique features that have been designed to increase comfort, safety, and ease of use. Product enhancements such as polished eyelets, easy-open packaging, and coude tips with control stripe, are among the many user driven innovations found in Cure Medical products.

### Cure Nation


Cure Medical offers a unique online and social media network designed to provide assistance and education through a personal support program called Cure Nation.



Information on the Cure Nation page at [www.curemedical.com](http://www.curemedical.com), and on related social media pages, helps build community and serves as a platform to share life enhancing topics.

Sign up on the Cure Medical website to enjoy these benefits:

- Receive the free Cure Nation e-newsletter
- Download free Accessible Lifestyle and Travel books
- Share ideas with others in the Cure Nation community
- Meet the Cure Advocates who use Cure Medical catheters

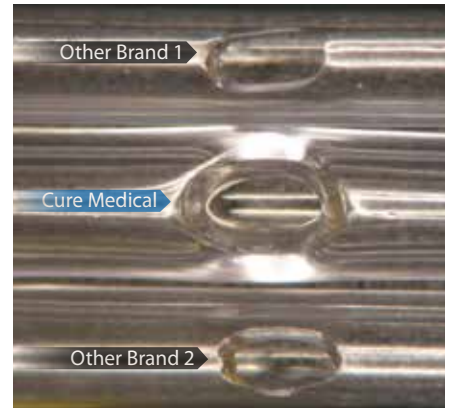

High magnification Scanning Electron Microscope (SEM) image comparing an eyelet on a Cure Medical catheter (middle) with two other national brand catheters (top & bottom).



Many Cure Catheter® packages have unique dual openings for increased ease of use.



The coude tip features a control stripe that helps enable proper use.

*“I've tried numbers of different catheters made by various manufacturers. The Cure Catheter has the best tip and is the easiest one for me to use.”*

– Able Outdoors Editor and quadriplegic Chad Waligura

# READY-TO-USE

Request free samples • [www.curemedical.com](http://www.curemedical.com) • 800.570.1778

## Cure Ultra®

The sterile, single use Cure Ultra is a ready-to-use catheter for men and women. The small package features an easy tear top with minimum waste material. The Cure Ultra features polished eyelets and a unique 'No Roll' connector/funnel end to help increase confidence when draining.



The Cure Ultra for men features a proprietary CoverAll™ application process for even distribution of lubricant as the catheter is removed from the package – making it ready-to-use with ultra-low resistance while cathing. Additionally, it includes a unique gripper for increased ease of use.

*Offered as a case of 300 units (10 boxes of 30).*

### Male

Part No.	Description	FR Size
ULTRA M8	Male, 16", Straight Tip	8
ULTRA M10	Male, 16", Straight Tip	10
ULTRA M10C	Male, 16", Coude Tip	10
ULTRA M12	Male, 16", Straight Tip	12
ULTRA M12C	Male, 16", Coude Tip	12
ULTRA M14	Male, 16", Straight Tip	14
ULTRA M14C	Male, 16", Coude Tip	14
ULTRA M16	Male, 16", Straight Tip	16
ULTRA M16C	Male, 16", Coude Tip	16
ULTRA M18	Male, 16", Straight Tip	18
ULTRA M18C	Male, 16", Coude Tip	18



The small, pre-lubricated Cure Ultra for women is highly portable and ready-to-use.

*Offered as a case of 300 units (10 boxes of 30).*

### Female

Part No.	Description	FR Size
ULTRA8	Female, 6", Straight Tip	8
ULTRA10	Female, 6", Straight Tip	10
ULTRA12	Female, 6", Straight Tip	12
ULTRA14	Female, 6", Straight Tip	14
ULTRA16	Female, 6", Straight Tip	16

“The Cure Ultra ready-to-use catheter for women is small and comes in a package that is bendable, thus allowing a person to slip one in their pocket easily. Having the lubrication built in is a convenience.”

– Accessible Home Expert and wheelchair user Rosemarie Rossetti

“The big advantage of the Cure Ultra catheters I use is that they are pre-lubricated. Cure Ultra truly has changed my life for the better.” – Cure Medical customer Wesley Cumpston



## Cure Twist®

The unique, sterile, single use, Cure Twist is similar to the design of cosmetics – so it is the ideal choice for women who value discretion and convenience. The Cure Twist features an easy, twist open top and polished eyelets on a pre-lubricated straight tip that allows for ‘No Drip/No Mess’ cathing. It is offered as a Single and as a Kit.



The Cure Twist is available as a Single with a universal funnel end for maximum compatibility with drainage solutions.

*Offered as a case of 300 units (10 boxes of 30).*

### Female, Single

Part No.	Description	FR Size
T8	Female, 6", Straight Tip	8
T10	Female, 6", Straight Tip	10
T12	Female, 6", Straight Tip	12
T14	Female, 6", Straight Tip	14
T16	Female, 6", Straight Tip	16



Ready-to-use Cure Twist with Insertion Kit supplies

The Cure Twist is available with Insertion Kit supplies including a BZK wipe, lubricant, gloves, underpad, and 1500 ml collection bag with universal connector.

*Offered as a case of 90 units (3 boxes of 30).*

### Female, Kit

Part No.	Description	FR Size
T8K*	Female, 6", Straight Tip w/Insert. Kit	8
T10K*	Female, 6", Straight Tip w/Insert. Kit	10
T12K*	Female, 6", Straight Tip w/Insert. Kit	12
T14K*	Female, 6", Straight Tip w/Insert. Kit	14
T16K*	Female, 6", Straight Tip w/Insert. Kit	16

“I am loving the Cure Twist. It’s so easy to use and throw away. And, besides, the packaging is pretty!”

– Ms. Wheelchair New York 2015 and NYC’s first registered nurse who uses a wheelchair Andrea Dalzell

\*Not available in Canada.

# DISCREET PORTABILITY

Request free samples • [www.curemedical.com](http://www.curemedical.com) • 800.570.1778

## Cure Medical® Pocket Catheter



The sterile, single use, U-shaped Cure Medical Pocket Catheter offers discreet portability by easily fitting in most pant pockets. It is offered with polished eyelets on a straight or coude tip. The Cure Medical Pocket Catheter is available in a standard (16") and an extra long (25") length with funnel end. It is offered as an uncoated Single plus an uncoated Kit and hydrophilic Kit.

Cure Medical Pocket Catheter configurations include:

- Single (U)
- Single with lubricant (UL)
- Extra long (XL) with lubricant and gripper sleeve
- Kit (UK) containing a nonstaining BZK wipe, one packet of lubricant, ambidextrous gloves, an underpad, and a 1500 ml collection bag

*Offered as a case of 300 units (10 boxes of 30).*



Part No.	Description	FR Size
M12U	Male, 16", Straight Tip	12
M12ULC*	Male, 16", Coude Tip with Lubricant	12
M14U	Male, 16", Straight Tip	14
M14XL*	Male, 25" (Extra Long), Straight Tip with Lubricant and Gripper Sleeve	14
M14UL*	Male, 16", Straight Tip with Lubricant	14
M14ULC*	Male, 16", Coude Tip with Lubricant	14
M16U	Male, 16", Straight Tip	16
M16ULC*	Male, 16", Coude Tip with Lubricant	16

*Offered as a case of 90 units (3 boxes of 30).*

Part No.	Description	FR Size
M14UK*	Male, 16", Straight Tip, Kit	14

“Until I found the extra long catheter from Cure Medical, I was stuck at home. The M14XL changed my life. The extender tube on the catheter is amazing, and is easy to use.” – Jenn Sexton Cure Medical customer

“I really appreciate that Cure Medical catheters are not made with scary chemicals like DEHP/DINP, BPA or latex.” – Adaptive Ski Instructor Chris Collin

\*Not available in Canada.

# HYDROPHILIC CATHETERS

Request free samples • [www.curemedical.com](http://www.curemedical.com) • 800.570.1778

## Hydrophilic Cure Catheter®

The sterile, single-use hydrophilic Cure Catheter features polished eyelets and a gripper sleeve to help ensure controlled, clean use. It hydrates within the package for mess-free/stain-free use (see the Simple Hydration & Prep Steps on page 8). The hydrophilic Cure Catheter for men is offered as a Single and with Insertion Kit supplies including BZK wipe, gloves, underpad, and a 1500 ml collection bag.



*Offered as a case of 300 units (10 boxes of 30).*



*Offered as a case of 300 units (10 boxes of 30).*

### Male, Single

Part No.	Description	FR Size
HM12	Hydrophilic, Male, 16", Straight Tip	12
HM12C	Hydrophilic, Male, 16", Coude Tip	12
HM14	Hydrophilic, Male, 16", Straight Tip	14
HM14C	Hydrophilic, Male, 16", Coude Tip	14
HM16	Hydrophilic, Male, 16", Straight Tip	16
HM16C	Hydrophilic, Male, 16", Coude Tip	16



*Offered as a case of 90 units (3 boxes of 30).*

### Male, Kit

Part No.	Description	FR Size
HM12UK*	Hydro., Male, 16", Straight Tip, Kit	12
HM14UK*	Hydro., Male, 16", Straight Tip, Kit	14
HM16UK*	Hydro., Male, 16", Straight Tip, Kit	16

\*Not available in Canada.

### Pediatric

Part No.	Description	FR Size
HP8	Hydrophilic, Pediatric, 10", Straight Tip	8
HP10	Hydrophilic, Pediatric, 10", Straight Tip	10
HP12	Hydrophilic, Pediatric, 10", Straight Tip	12
HP14	Hydrophilic, Pediatric, 10", Straight Tip	14



### Pediatric Resources

Cure Medical offers a unique Education Program to teach 'How To Cath' using a backpack full of fun, instruction resources such as flashcards and coloring books.

Request a complimentary backpack with education resources for your child at [www.curemedical.com](http://www.curemedical.com).

“My daughter is the first child with spina bifida at her school, so we needed to educate the staff. The pediatric materials in Cure Medical's backpack have been helpful for EVERYONE.” – Cure Medical customer Amanda Kerr

See Simple Hydration & Prep Steps on the next page ►

# CATH IN A BAG

Request free samples • [www.curemedical.com](http://www.curemedical.com) • 800.570.1778

## Simple Hydration & Prep Steps for the Hydrophilic Cure Catheter

The hydrophilic Cure Catheter (page 7), easily hydrates within the package for mess-free/stain-free use by following these simple steps:

- 1 Fold or press on the packet to release the purified water into the package as shown.



Option A: FOLD the packet until water flows



Option B: PRESS on the packet until water flows

- 2 When ready to use, open the end of the package just enough to use the Gripper Sleeve to remove the catheter – keeping the water in the package.
- 3 Peel off the round, white backing to stick the catheter package to a vertical surface.
- 4 For best results, wait at least 30 seconds for the wet, hydrophilic coating to lubricate the catheter.

## Cure Catheter® Closed System (Single)



The sterile, single-use, unisex Cure Catheter Closed System Single features polished eyelets on a pre-lubricated catheter tip with integrated 1500 ml collection bag.

*Offered as a case of 100 units.*

### Unisex, Single

Part No.	Description	FR Size
CB8	Closed System Single, Straight Tip	8
CB10	Closed System Single, Straight Tip	10
CB12	Closed System Single, Straight Tip	12
CB14	Closed System Single, Straight Tip	14
CB16	Closed System Single, Straight Tip	16

“I like the hydrophilic Cure Catheter with straight tip because – from a quality standpoint, compared to some of the other catheters I’ve used, it has been the most comfortable and the easiest to use.”

– Triumph Foundation Peer Counselor Andrew Angulo

“The Cure Catheter Closed System is easier to use and draining is more complete. Thank you for everything you do.”

– Cure Catheter Closed System user



## Cure Catheter® Closed System (Kit)



The sterile, single-use, unisex Cure Catheter Closed System Kit features polished eyelets on a pre-lubricated catheter tip with integrated 1500 ml collection bag. It is offered with options for straight or coude tip design.

The Kit includes the Cure Catheter Closed System, a nonstaining BZK wipe, povidone-iodine swabsticks, ambidextrous gloves, gauze wipe, and an underpad.

*Offered as a case of 100 units.*

### Unisex, Kit

Part No.	Description	FR Size
CS8*	Closed System Kit, Straight Tip	8
CS10*	Closed System Kit, Straight Tip	10
CS12*	Closed System Kit, Straight Tip	12
CS14*	Closed System Kit, Straight Tip	14
CS14C*	Closed System Kit, Coude Tip	14
CS16*	Closed System Kit, Straight Tip	16

“I like the pre-lubricated Cure Catheter Closed System, because it doesn’t kink-up or collapse. Although I’ve used several different brands of catheters since my injury, I’ve found Cure Medical catheters to be the easiest to use and I’ve had the fewest problems with them. I’m sold on Cure Medical catheters.”

– WCMX Star Jerry Diaz

\*Not available in Canada.

## CONVENIENT OPTIONS

Request free samples • [www.curemedical.com](http://www.curemedical.com) • 800.570.1778

### Cure Medical® Catheter Insertion Kit

When combined with a catheter (not included), the sterile, single-use, unisex Cure Medical Catheter Insertion Kit is an alternative to using a complete Closed System product. Insertion Kits include a nonstaining BZK wipe, one packet of lubricant, ambidextrous gloves, an underpad, and one of two bag options.



*Offered as a case of 100 units.*

Part No.	Description
K1	Insertion Kit with resealable disposal bag



*Offered as a case of 90 units.*

Part No.	Description
K2-90	Insertion Kit with universal connector on a 1500 ml collection bag



*Offered as a case of 100 units.*


Part No.	Description
K2-100	Insertion Kit with universal connector on a 1500 ml collection bag

### Cure Medical® Extension Tube

The non-sterile, flexible Cure Medical Extension Tube easily connects to any catheter to facilitate draining.

*Offered as a case of 30 units.*

Part No.	Description
ET1	29" tube

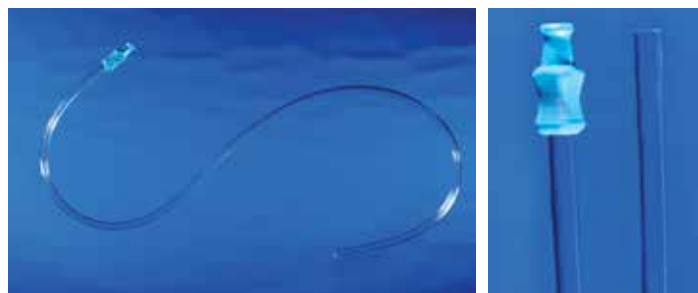

## Cure Catheter®

The sterile, single-use Cure Catheter features polished eyelets on a straight or coude tip with control stripe. It is offered in male, female, and pediatric lengths. In addition to uncoated, classic designs, many Cure Catheters are available with a hydrophilic coating (see page 7). Cure Catheter packages feature unique, dual openings for increased ease of use.



*Offered as a case of 300 units (10 boxes of 30).*

### Male, straight tip

Part No.	Description	FR Size
M8	Male, 16", Straight Tip	8
M10	Male, 16", Straight Tip	10
M12	Male, 16", Straight Tip	12
M14	Male, 16", Straight Tip	14
M16	Male, 16", Straight Tip	16
M18	Male, 16", Straight Tip	18

### Male, coude tip

Part No.	Description	FR Size
M10C	Male, 16", Coude Tip	10
M12C	Male, 16", Coude Tip	12
M14C	Male, 16", Coude Tip	14
M16C	Male, 16", Coude Tip	16
M18C	Male, 16", Coude Tip	18

*Offered as a case of 300 units (10 boxes of 30).*

### Female

Part No.	Description	FR Size
F8	Female, 6", Straight Tip	8
F10	Female, 6", Straight Tip	10
F12	Female, 6", Straight Tip	12
F14	Female, 6", Straight Tip	14
F14NC	Female, 6", Straight Tip, No Connector	14
F16	Female, 6", Straight Tip	16
F18	Female, 6", Straight Tip	18

### Pediatric

Part No.	Description	FR Size
P8	Pediatric, 10", Straight Tip	8
P10	Pediatric, 10", Straight Tip	10
P12	Pediatric, 10", Straight Tip	12
P14	Pediatric, 10", Straight Tip	14

“I find Cure Catheters to be a far superior product to what I'd been using!”

– Abilities Expo host and Transverse Myelitis survivor Julianne Dallara



## Better Today. Brighter Tomorrow.

Cure Medical products are quality-made at a reimbursement-friendly cost, and they offer premium features that users immediately enjoy. We understand spinal cord injuries (SCI) and central nervous system disorders (CNS/D) personally so we work hard to design our products with your need in mind. At Cure Medical, we have a vested interest in the quality, safety, comfort, and convenience of our intermittent catheters – as our founder's health depends on them too.



## Research Is The Key.

Only Cure Medical has made an unsurpassed Cure Commitment to donate 10% of net income to multiple medical research facilities nationwide in pursuit of a cure for spinal cord injuries and central nervous system disorders. For information on advances made in medical science, visit [www.curemedical.com](http://www.curemedical.com).

“Visit [www.curemedical.com](http://www.curemedical.com) for product information, research updates, and to sign up for the Cure Nation – a community of friends who use catheters. Cure Nation members enjoy free resources, a monthly e-newsletter, and lifestyle conversations with Cure Advocates.”



Cure Medical  
3471 Via Lido, Suite 211, Newport Beach, CA 92663

phone 800.570.1778 or 949.723.0364  
fax 949.723.0564 • [www.curemedical.com](http://www.curemedical.com)

Follow us on



Cure Medical, the Cure logo, Cure Catheter, Cure Twist, and Cure Ultra are registered trademarks of Cure Medical LLC.